

# SHIPPING INSTRUCTIONS



## 1. Genital + Extragenital specimen collection kit:

- Follow Genital Collection Instructions (or Extragenital Collection Instructions if you purchased genital, oral and rectal testing) to collect specimen
- Seal transport tube(s) in one of the biohazard bags with absorbent pad


Applicable tests: Chlamydia/Gonorrhea, Trich, HPV, Mycoplasma/Ureaplasma

## 2. Blood sample collection kit:

- Follow the sample collection instructions in the blood kit (white box) to collect the specimen
- Double check that all circles are completely filled to the line.**
- Incomplete blood spots may be rejected by the lab.**
- Seal blood card in one of the biohazard bags with desiccant pack and absorbent pad

Applicable tests: HIV, Syphilis, Herpes Simplex II, Hepatitis C

- Place the sealed biohazard bag(s) in the box and seal the box
- Complete lab requisition form
- Place the sealed box and complete lab requisition form in the pre-paid USPS return envelope or clinical pak and seal it
- Ship package to the lab from any USPS pickup location or place in your mailbox
- You will receive an email approximately 2-3 days after your sample is received by the lab to notify you that your results are posted to your account. If you do not receive an email in 7 days, you may login to your account any time to view results.

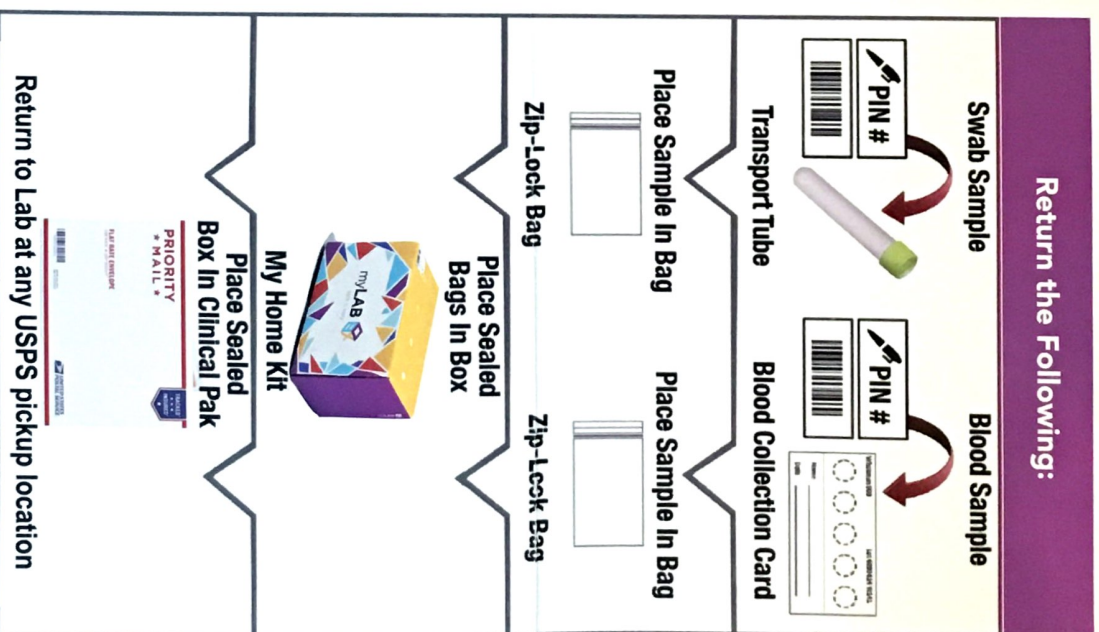

myLAB Box customers who test positive are eligible for complimentary telemedicine consultations with a physician. To request your consultation, contact [customerservice@mylabbox.com](mailto:customerservice@mylabbox.com)